

homewave

Home Prescription
Neuromodulation
Pain Therapy System

Patient Brochure

biowave

Manufactured by
Biowave Corporation
16 Knight Street
Norwalk, CT 06851

1-877-BIOWAVE x2

1-877-246-9283 x2

info@biowave.com

www.biowave.com



The Pathway to Pain Relief

The Pathway to Pain Relief

The Homewave® Neuromodulation Pain Therapy System utilizes a patented technology to deliver electrical signals through the skin into deep tissue for inhibiting pain transmission and improving function.

Homewave® can be used to relieve pain in the lower back, cervical spine, buttocks, hip, groin, knee, ankle, foot, neck, shoulder, elbow, wrist, hand and fingers.

Facilitate and Accelerate Physical Therapy

Homewave is also used to facilitate and accelerate the exercise portion of physical therapy. By using Homewave first, in place of heat, you can obtain a significant long acting period of pain relief as well as a functional improvement including an increase in range of motion. The exercise portion of physical therapy should then be conducted immediately following the Homewave® treatment. The result is that you can move more easily through a greater range of motion, the whole key to getting better faster. Plus there is limited post exercise pain.

Advanced Deep Tissue Technology

The Homewave® System utilizes a proprietary signal mixing technology that delivers two continuously summed, high frequency alternating current signals that pass through the skin into deep tissue where they interact with tissues to produce a range of signals which include a low frequency electrical field. This field is optimized for blocking pain and providing functional improvement including a greater range of motion and a reduction in stiffness and muscle spasm. In addition to providing efficacy, these signals are significantly more comfortable leading to improved patient treatment compliance.

Additionally, Homewave utilizes SmartPAD™ technology in its electrodes. The SmartPAD™ electrodes allow for the continuous monitoring and automatic adjustment of the electrical signal technology ensuring the accurate and safe delivery of therapeutic energy into deep tissue.

The Homewave® Neuromodulation Pain Therapy System received 510(k) clearance from the US. Food and Drug Administration with approval granted for relief of chronic pain, acute pain and postoperative pain.

Proven Clinical Efficacy

Clinical studies have shown that following a 20 minute treatment, benefits include a significant reduction in pain and an improvement in function including an increase in range of motion, decrease in muscle spasm and decrease in stiffness, all of which last for up to 24 hours post treatment, without the side effects that are common to drugs, injection therapies, surgery and implanted devices.

Easy to Use

There are just 3 buttons that operate the Homewave Device and there is no programming. The signals are optimized for penetrating the skin, blocking pain and improving function.



Ask Your Doctor

Ask your doctor about the Homewave® Neuromodulation Pain Therapy System. Your doctor can write you a prescription for Homewave®.

Contact Us

Additional information is available at:

www.biowave.com

1-877-BIOWAVE x2

(1-877-246-9283 x2)



The Pathway to Pain Relief